

Portable Contact Lens Automatic Ultrasonic Cleaner





1. Upper cover
2. Main body
3. Charging port
4. Non-slip mat
5. Bottom drainage outlet



1. Makeup mirror
2. Dual eyeglass frame
3. Switch
4. LED display screen
5. Suction rod placement slot
6. Tweezers placement slot
7. Time

Specifications

- Model: YQ-066
- Rated charging voltage: 5V
- Rated charging current: 1A
- Rated power: 2W
- Weight: about 85g
- Size: 100x65x36mm
- Execution standards: GB4706.1-2005
- Contains: Host, tweezer suction rod, dual eyeglass frame, charging cable

Operating instructions

Glasses cleaning



1. Open the upper cover of the cleaning box
2. Open the cleaning tank cover and place the contact lens in it
3. Pour in contact lens care solution

Note: Please distinguish left and right when placing

Note: The care solution should be immersed in the lenses




4. Cover the cleaning tank cover, press the start button to start, press t to adjust the time, and it will automatically end
5. After cleaning, remove the dual cleaning box and take out the lenses
6. Pour out the liquid and clean the double box

The contact lens cleaning box will automatically end working according to the time. If water accidentally spills out during operation, the bottom drainage port will discharge the waste liquid.

Note: Tighten the cleaning cover to prevent leakage

Machine charging

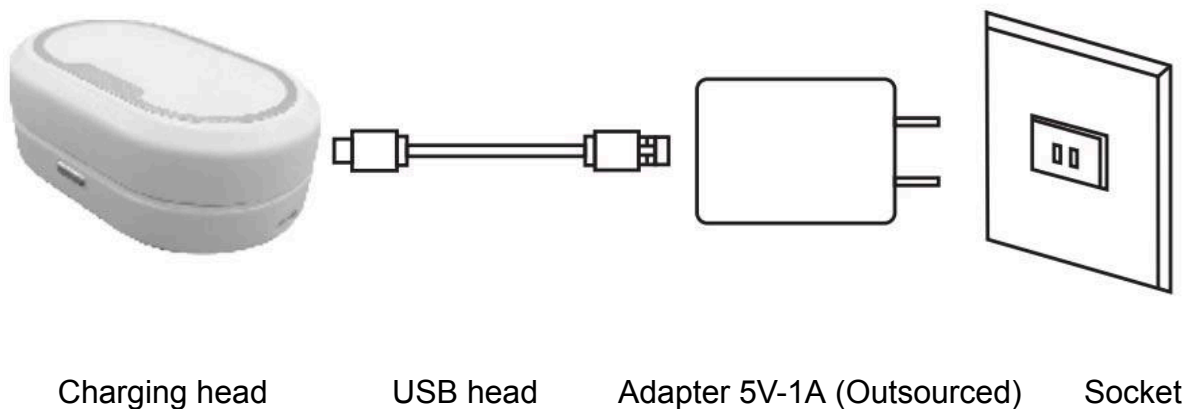

Note: Charging with a large adapter may cause damage to the machine battery. It is recommended to use a 5V/1A charging adapter.

When the machine's battery is low, the LED display screen shows LO. At this time, please plug in the charging cable and connect the adapter.

When the power is charged, the screen displays three horizontal bar cycles in the charging state. After the charging is completed, the display screen displays FU.